

Use of Portable Ramps for Wheelchair Access

- The portable ramps are located in the cellar behind the bar area at Pen-y-Pound
- They are very strong and hence heavy (20kg each) It is recommended that 2 people are involved when carrying them. A green sack trolley is available to transport them if required.
- These have been modified to fit either side of the threshold step of the main entrance.
- The one with slots cut out is for the inside, and should be installed such that the deepest face is flush with the threshold step as shown below.
- The other ramp is slid between the shutter uprights on the outside of the main entrance, and should be installed such that the deepest face is flush with the threshold step as shown below.
- Removal and replacement to the cellar is the reverse procedure.

