


IPAD SP1 Semi-auto AED

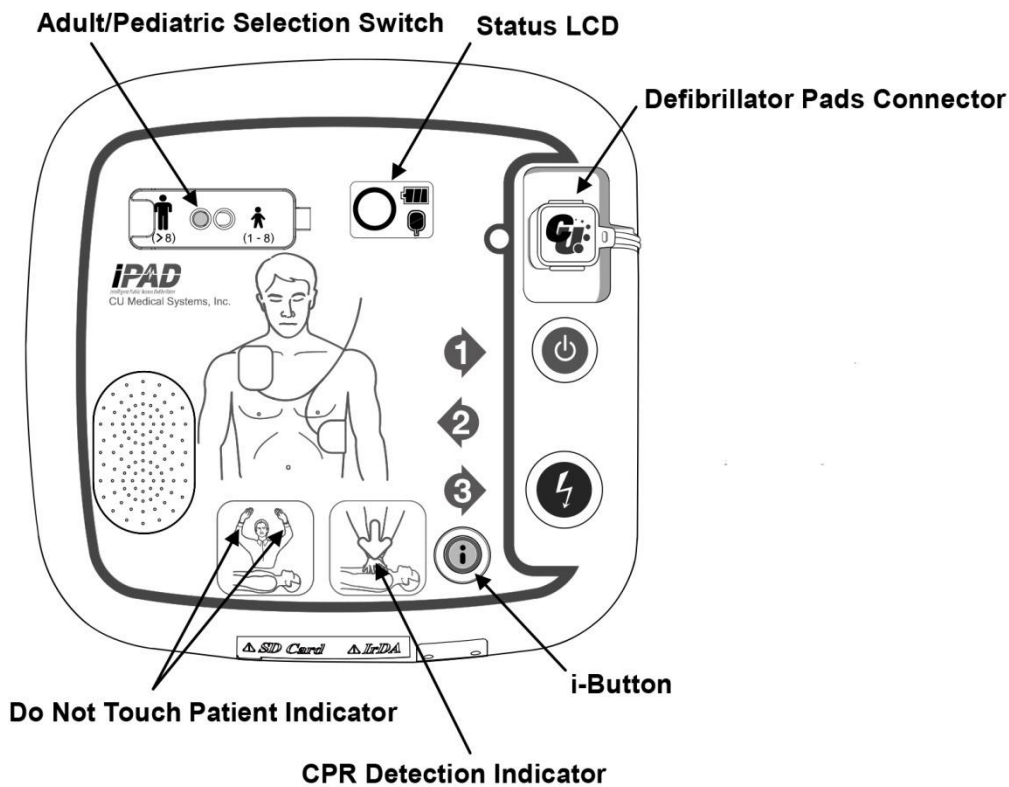

Adult/Pediatric Selection Switch

Selects Adult/Pediatric modes.
Lift cover & slide switch. (Adult by default).
Use Pediatric mode for child 1-8yrs, less than 25Kg.

Status LCD

Displays the current status of the device, battery and pads.
● ready : X error – remedy or get another device.

Defibrillator Pads Connector

Pad connection socket. Ensure plug securely inserted.

i-Button

Shows Red when device in error. (Get another device asap).

Flashes Blue after delivering shock – press if CPR voice prompts required.

Do Not Touch Patient Indicator

Flashes to warn NOT to touch the patient.