


# IPAD SP1 Semi-auto AED

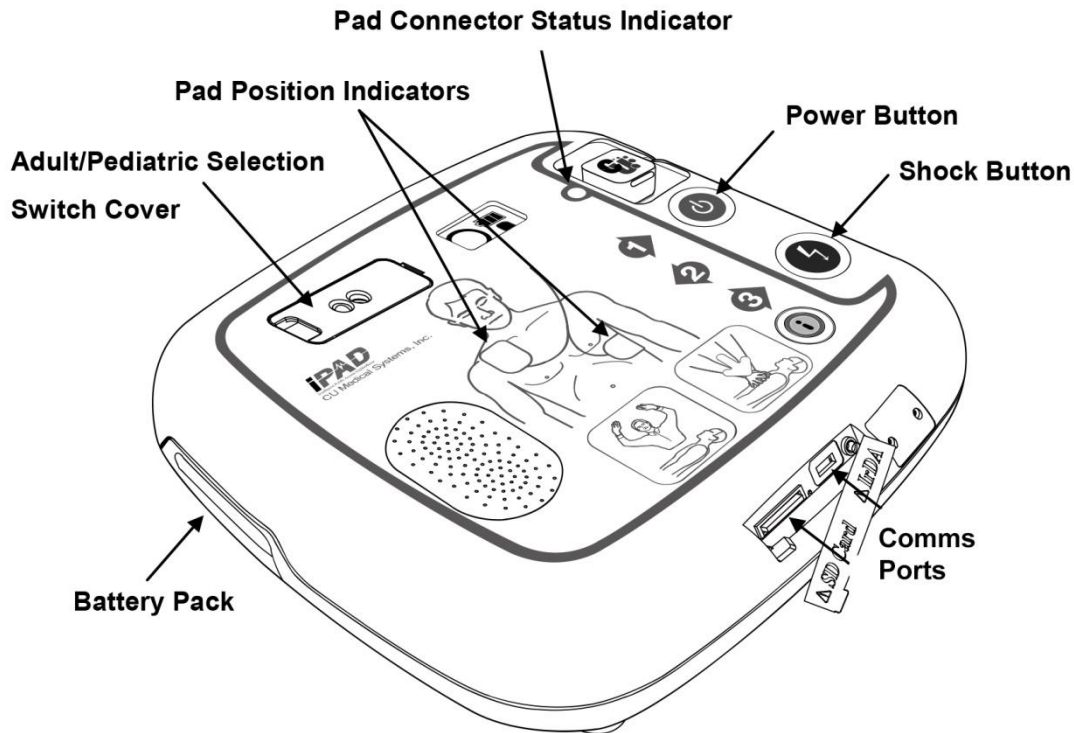

## Power Button

Turns the device on/off. Green LED lit when on (hold for one second to switch off).

## Shock Button

Continuous Beep & flashing Orange when ready to shock. Delivers defibrillating shock when pressed. *(If not done in 15 sec. cancels shock and calls for CPR)*

## Pad Connector Status Indicator

Steady green when pads correctly connected. Re-plug if flashing.

## Pad Position Indicators

Indicates the pad position on adult patient.

## Adult/Pediatric Selection Switch Cover

Lift cover & slide switch. Selects Adult/Pediatric modes. Adult by default. Use Pediatric mode for child 1-8yrs / less than 25Kg.

## WARNINGS

Remove Necklaces and any other metal that could come in contact, or near, the applied pads.

Dry any wet skin prior to applying pads. Use wipes to cleanse if necessary.

Unless very severe amount of chest hair ignore razor & shaving.

Disconnect any medical equipment before 'shocking'.